

- **The Straps**

There are a variety of straps available for use with the Goodwood (pelvic & lap, groin, chest and foot). They are used for security and to improve the child's position whilst seated and should always be used. Once the child is in the chair, secure them with the straps before you do anything else. All straps are securely fastened to the chair.

- **The Pommel**

Optional sized pommels are available with a quick release pommel mount. Pommels are supplied fitted to the chair at point of sale.

- **Hi-Lo Frame**

The chair must be securely attached to the Hi-Lo frame

Refer to user instructions for the Hi-Lo Frame.

Ensure that the chair is SECURELY ATTACHED to the frame before allowing it to be used.



The Goodwood Integra Chair



USER GUIDE

IMPORTANT

This chair is very versatile and has been developed to meet the needs of children with a wide range with special needs. There are many special adaptations available.

If you require replacement parts or you are in any doubt as to the continued safe use of your Goodwood chair, please do not hesitate to contact our customer services department on the telephone number shown below.



DCS Jancare
7/8 Radley Place, Radley Rd. Ind. Est.
Abingdon, Oxon. OX14 3RY



Tel: 01235 523353 Fax: 01235 531019

A
DCS Jancare
Brand Product

IMPORTANT- READ THESE INSTRUCTIONS BEFORE USE: Failure to follow these instructions, especially with regard to moving parts, may pose a risk to the user.

- **Before Using Your Goodwood Chair**

Assembly of parts/products must be done before use. Should you have any queries regarding this product, please address them before use. **DCS JONCARE recommend that a therapist be consulted when adjusting any supports.** The product should be used only on a level floor.

- **Maintenance**

Keep products well maintained at all times. Clean products regularly using a non-abrasive cleaner to ensure maximum life. Note any changes in performance and contact DCS JONCARE for advice if necessary. It is recommended that the product is checked weekly to ensure that all fixings and bolts are securely attached and tightened.

- **Precautions for Chairs Fitted With Castors**

This product is NOT to be used as a mobility aid. Castors are fitted to ease basic transportation - within a room having a flat surface. Apply brakes when stationary. **Do not** push the product along when the brakes are applied, **Do not** push the product by its headrest, **Do not** lean or put excessive weight on the tray.

- **Safety**

All safety straps fitted to the chair must be employed when the product is in use. **Do not** leave children unattended. **Do not** allow siblings or other children to climb onto the equipment. This product contains small parts - take care with young children.

GOODWOOD SIZE 2 MAXIMUM USER WEIGHT 60KG
GOODWOOD SIZE 3 MAXIMUM USER WEIGHT 65KG
GOODWOOD SIZE 4 MAXIMUM USER WEIGHT 70KG
GOODWOOD SIZE 5 MAXIMUM USER WEIGHT 85KG
GOODWOOD SIZE 6 MAXIMUM USER WEIGHT 85KG

- **If There is a Fault With Your Chair**

Do not use or allow others to use the product if you suspect that it is faulty or broken. Contact DCS JONCARE immediately for advice. Remember - we're here to help you if you need it! **If you do have any problems, please call us on 01235 523353.**

Issue August 2015

- **Use With Other Manufacturer's Products**

This product conforms to the Medical Devices Directive. It has not been tested in conjunction with other manufacturer's products. **Do not** use accessories other than DCS JONCARE accessories that have been designed specifically for use with this product.

Weight of product:

Size 1 = 3.1 kg, Size 2 = 4.9kg, Size 3 = 6.8kg, Size 4 = 8.7kg
Size 5 = 10.6kg, Size 6 = 12.5kg.

THE INITIAL SET UP OF YOUR GOODWOOD CHAIR

Your chair will arrive set up to the agreed specifications at the time of the assessment. If the child's needs have changed, please refer to the "ADJUSTMENT" section. Accessories will usually be fitted to the chair before despatch. However, for packing purposes, trays and head supports may be wrapped separately.

PLEASE MAKE SURE YOU HAVE RECEIVED ALL THE PARTS ORDERED BEFORE USING THE CHAIR (INCLUDING THE ALLEN KEY USED TO MAKE ADJUSTMENTS).

Making Adjustments to Your Goodwood Chair

- **Seat Depth**

The seat itself can be adjusted forward or back by 50mm (2") by repositioning the fixing bolts located underneath the seat. Alternatively, you can use the variety of padded back supports available (flat or corner shaped).

- **Seat Width**

This can be adjusted by using the variety of padded width reduction pads available (parallel or tapered). These are attached to the chair using Velcro.

- **Lateral Support**

Scolio pads are available if additional support is needed. These are adjustable in height using the Allen key (supplied) and in depth by using spacers. Always retighten the bolts securely after making adjustments.

- **Head Support**

The head support can be adjusted up or down. Loosen the bolts using the Allen key (supplied), slide the support to required position and retighten the bolts securely.

- **Armrests**

The height and angle of the armrests can be adjusted, or they can be completely removed if you wish the chair to be used in conjunction with a table.

1. **To adjust the height**, remove the bolts located inside each chair leg, using the Allen key (supplied). Relocate the armrests as required, insert the bolts and re-tighten them securely.
2. **To adjust the angle**, remove the armrests as above. Select the appropriate holes to achieve the desired angle. Insert the bolts and re-tighten them securely.
3. **To remove the armrests**, remove the bolts located inside each chair leg. Take the arm rests off and keep the securing bolts in a safe place for future use. **Remember to keep small objects from the reach of infants and young children.**

- **The Footshoes**

These are attached to the footboard from the top and can be adjusted forward or back. To adjust the footshoes, loosen the bolts using the Allen key (supplied). Select the desired position and retighten the bolts securely.

- **The Tray**

1. **To attach the tray to the chair**, slide it carefully onto the chair arms pulling the black plungers down as you do so. Push the tray toward the back of the chair until the plungers click into the locating holes provided under the armrests.
2. **To adjust the tray depth**, pull down on the plungers sliding the tray forward or back to the desired position. Make sure each plunger clicks into the nearest locating holes to the position chosen.