

RehaNorm Bingen GmbH



User manual

MTS

Mannheimer Toilet and

Shower Chair

Dear customers,

Congratulations on purchasing your bath chair.

Please read all the information contained in this manual before first using the bath chair, note the information about a safe handling and retain it for future reference.

Please read all the information contained in this manual before first using the bath chair!

EEC Conformity Declaration:

We as the producer declare in our exclusive responsibility that the bath and shower chair performs the basic requests of the EG-direction concerning medical products 93/42/EWG, attachment 1.

RehaNorm Bingen GmbH

D-55411 Bingen – Am Ockenheimer Graben 50

E-Mail: information@rehanorm.de

Internet: www.rehanorm.de

Tel: ++49 6721 9060 - Fax: ++49 6724 9060

August 2010

Contents page:

<i>Important safety information first.....</i>	<i>4</i>
<i>1. Application reach.....</i>	<i>5</i>
<i>2. Delivery.....</i>	<i>5</i>
<i>3. Adjustments.....</i>	<i>6</i>
3.1 <i>The seating system.....</i>	<i>6</i>
3.2 <i>The footplate.....</i>	<i>7</i>
3.3 <i>The head rest.....</i>	<i>7</i>
3.4 <i>Applying brakes.....</i>	<i>8</i>
3.5 <i>Safety handle and toilet basin.....</i>	<i>8</i>
3.6 <i>Arm rests.....</i>	<i>9</i>
3.7 <i>Accessories.....</i>	<i>9</i>
<i>4. Cleaning, Care and Maintenance Instructions.....</i>	<i>10</i>
<i>5. Technical facts.....</i>	<i>10</i>
<i>6. Guarantee.....</i>	<i>10</i>

Important safety information first:

- **Check regularly and always after adjustments that all screws are in good condition and fitted securely.**
 - **Don't use the bath chair in the open air.**
 - **Use the bath chair only on solid, plain and horizontal ground.**
 - **Avoid leaning out of the bath chair. There is the danger of falling.**
 - **Do never step on the footplate to get in or out of the chair. There is the danger of falling.**
 - **Put down the chair only when you've made sure that the brakes are applied.**
 - **Apply the brakes before you get in or out of the chair.**
 - **When it is necessary to transport the chair above a hindrance like a stair within the patient, the available establishments like an approach or an elevator should be used.**
 - **If there are no such establishments, the hindrance should be subdued by at least two assistants carrying the chair. For that purpose they seize the chair only at its solid steel frame. Never lift the chair at its seating system.**
- **In spite of its anatomical form the bath chair is not developed for a longer sitting. When the using of the toilet is concluded, it should be changed to another sitting- or lying-possibility.**
- **The construction of the bath chair allows a simple use. The toilet basin can be taken out to the back and is pushed under the seating system from there, too. Pay attention to the good fit of the toilet basin! You can also convey the bath chair directly above the installed toilet basin of your bath rooms. Take care that the height of the toilet basin and the bath chair are adapted well and never forget to apply the brakes.**

1. Application reach

The bath chair is applicanted in the domestic and in the hospital reach. It serves patients needing care with a limited radius of movement and/or limited control of their body-functions at the use of the toilet. The weight limit of the bath chair is a maximum user weight of 140 kg.

2. Delivery

The delivery contents a ready assembled bath chair with Ortho-Dynamic-Seating system out of reinforced plastic, detachable toilet basin and 75mm castors inclusive brakes.

Further accessories are already added corresponding to your order.

This manual belongs to the product, too and should be retained for future reference.

3. Adjustments


All parts should only be adjusted with an empty chair without the user in it. Please check all the connections afterwards.

3.1 Seating system

The seating system is adjustable in the height and in tilt in space.

To adjust the tilt in space loosen the nuts on the right and left side of the frame tube.

The tilt in space can now be adjusted corresponding to the teeth in the holding plate in little steps. Regularly the sit is adjusted in a horizontal position.

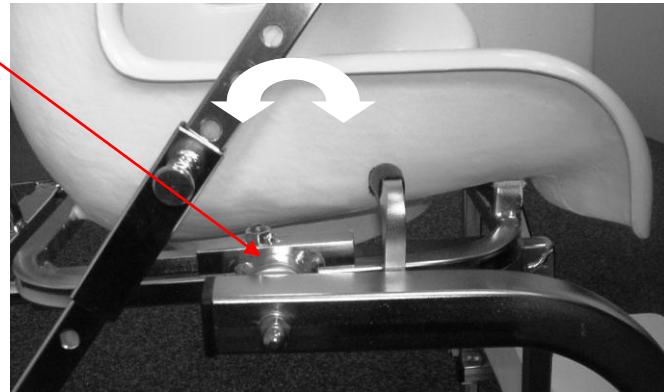

To adjust the height of the seat, loosen the wreath pins on the right and the left side of the main tube. The seat can be adjusted now. Pull out both tubes




equally and take care that none is pulled out more than the other.


Pay attention to a plug in depth of the tube of at least 10 cm.


After the adjustment all screws, nuts and pins have to be locked securely.


3.2 Footplate

The footplate can be adjusted in height and tilt in space.

To adjust the tilt in space please loosen the levers  on the right and the left side of the footplate tube.

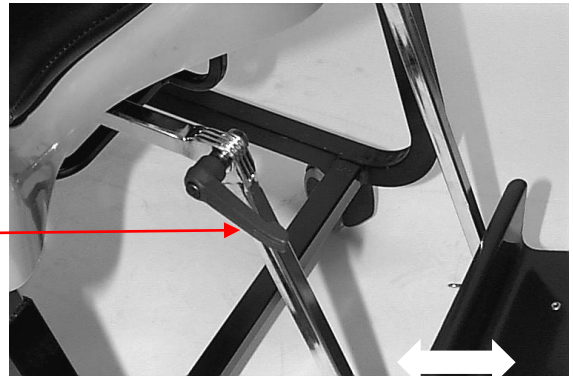

The height of the footplate can be adjusted after loosening the screws  on the footplate.

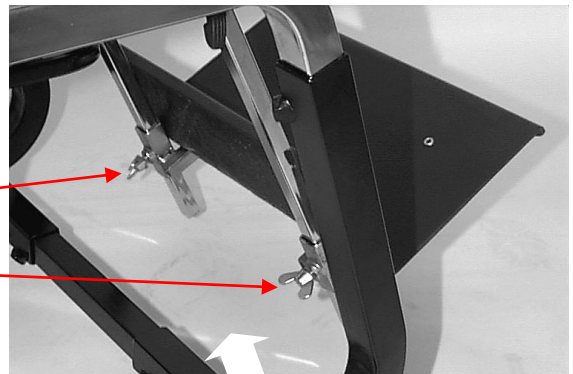


After the adjustment all screws, nuts and pins have to be locked securely.

3.3 Head rest

The head rest can be adjusted in height and in depth.

To adjust the depth please loosen the screw behind the head rest and adapt the depth to the needs of the user.

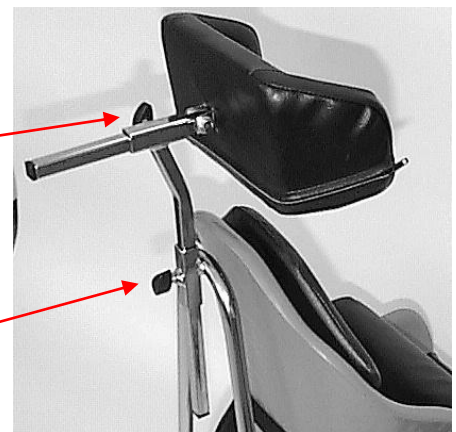

To adjust the height please loosen the screw behind the back support. Now push the headrest to the correct height.

After the adjustment all screws, nuts and pins have to be locked securely.


3.4 Applying brakes

Suppressing this lever with the foot will apply the brake to the castor
To remove the brake simply flick the lever back up using the foot..



3.5 Safety handle and toilet basin

The safety handle which should avoid the slide to the front can be put on to the tube or off the tube by simply pulling the handles on the right and the left side of the seat.
By simple loosening of the wreath pins the safety handle can be pushed to the front or the back.

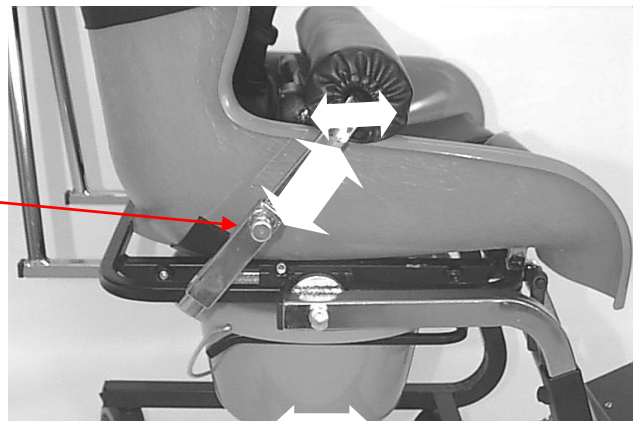

The toilet basin can be taken out by pulling it backwards. Pay attention that it is push just to the stop when it is inserted again.

After the adjustment all screws, nuts and pins have to be locked securely.

3.6 Arm rests

Beside the safety handle, arm rests which can be adjusted in height can be added to the bath chair.



After loosening the screws on the right and the left side of the tube, the arm rests can be adjusted in height free from hindrances and can be adapted to the needed position. Please fasten the screws afterwards.

3.7 Accessories

Furthermore there is the possibility to add handles for pushing the chair, which are adjustable in height and depth.

Please find the newest product information on our homepage www.rehanorm.de

4. Cleaning, Care and Maintenance Instructions

To clean the bath chair please use water and a mild detergent and, when necessary, an usual disinfectant on alcoholic base.

Do not use bleach or biological or phenolic based cleaners.

Rub off the metal parts with a dry cloth after the cleaning.

5. Technical facts

	size 1	size 2	size 3	size 4	size 5	size 6	size 7
Width of the seat, pelvis	18cm	24cm	28cm	34cm	38cm	43cm	43cm
Width of the seat, in front	28cm	33cm	39cm	45cm	53cm	58cm	60cm
Depth of the seat	20cm	24cm	30cm	36cm	38cm	40cm	47cm
Height of the back	30cm	37cm	46cm	53cm	60cm	62cm	77cm
Weight (without accessories)	9 kg	11 kg	13 kg	15 kg	17 kg	19 kg	21 kg

The change of the technical facts depends on the development of the product. We subject to change them.

Maximum weight of the user: 140 kg

6. Guarantee

We grant you a guarantee of 1 year to all parts of the bath chair when it is used correctly.